

NJ Phillips General Cleaning & Sterilisation Procedure



Cleaning Procedure



1. Flush the instrument and feed tube thoroughly with a warm water and detergent mix.



2. Re-flush with clean water.



3. Disconnect the feed tube and lubricate the instrument by pouring a small quantity of NJ Phillips Lubricant through the inlet fitting.

Sterilising Procedure



1. Attach the feed tube.



2. Pump the handle to draw water into the cylinder through the feed tube.



The image above is an example of what can happen when the instrument has **not been suspended** whilst in boiling water.



3. Suspend the complete instrument in a container of water, add the needles and boil for 10 to 20 minutes.



4. Remove the instrument from the container, wrap a cloth around the handle and pump dry.